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INTENDED USE: The Harmony® Latex & the Harmony® Polyisoprene* Dams are used as a barrier when engaging in oral/vaginal and oral/anal sex to help reduce the transmission of bodily fluids, harmful pathogens & sexually transmitted infections (STIs). External use only. Do not use during intercourse.

Always use a new Harmony® Dam each time you switch from oral/anal to oral/vaginal sex to reduce the risk of infection from harmful pathogens.

DIRECTIONS FOR USE: Open the package carefully. Do not use sharp objects, fingernails or teeth to open package. Remove dam from the package & lay it flat to cover the vagina & clitoris. When engaging in oral/anal sex cover the entire anus. When engaging in oral sex hold the edges of the dam to ensure it stays in place.

PRECAUTIONS: If using lubricant, use a proper water based personal lubricant. DO NOT USE oil-based products (petroleum jelly, mineral oil, baby oil, vegetable oil or similar substances) as they may damage the dam. DO NOT use the dam if the material is sticky, brittle or obviously damaged. DO NOT stretch the dam. DO NOT reuse or share a dam. Dispose of the dam after use in a waste container. Store in a cool, dry place & avoid excessive heat & direct sunlight. Consult a doctor or pharmacist about the compatibility of topical medicines that may come in contact with the dam.

When used properly, Harmony® Dams may help reduce the risk of transmission of many STIs; however, they cannot totally eliminate the risk.

Dams are for single use only. If the color is uneven or changed, do not use the dam.

*Not made with natural rubber latex.

CAUTION: The Harmony® Latex Dam contains natural rubber latex which may cause allergic reactions.

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